Difenser

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Manual English

Welcome to the manual for the hand disinfectant dispenser Difenser by Dilizer

We would like to thank you in advance for your trust in our products and will continue to try to make it as easy as possible for you to use them with explanations, instructions, but also tips and tricks.

Please share your experiences with our products with us, so we can improve these to your satisfaction.

Your Team from

DILIZER

Proper use

Please pay particular attention to the points which are provided with a safety note:

Non - observance can cause personal injuring and / or equipment damage!

-  • The appliance must not be operated with oils, greases, aggressive media, soaps, highly viscous liquids and beverages.
-  • Only approved disinfectant liquids be used for application.
-  • The Installation of the appliance must be implemented by building services, qualified personnel or independently with professional knowledge.
-  • Ensure that the room size and ventilation are adequate when choosing the installation location.
-  • Never use the appliance in potentially explosive areas.
-  • Make sure that the appliance is only used at a relative air moisture of <90% (not condensing).
-  • Please check whether it is necessary to upgrade the available firefighting measures.
-  • Only use this appliance at room temperatures between 05° – 45°C.

General safety

-  • No changes may be made to the appliance, otherwise the warranty will be void. Repairs to the appliance may only be carried out by authorised persons. Improper repairs can result in considerable dangers for which the manufacturer and distributor do not assume any guarantee.
-  • This appliance is not a toy. Keep and use it out of the reach of children. The appliance should be installed where children cannot reach it.
-  • Persons with physical, sensory or mental impairment should be supervised when using this appliance.
-  • The appliance is not suitable for the dosage of other than the indicated media.

Safety during operation of the appliance

-  • Never submerge the appliance into water.
-  • The appliance shouldn't be cleaned with water jets or by means of a high pressure cleaner.
-  • Only fill the appliance with the indicated substances in accordance with the intended use.
-  • Do not expose the appliance to thermal radiation and heat.

Disinfectant

- Please consider the safety data sheet of the disinfectant.
- If disinfectants other than those of the manufacturer/distributor are used, they do not assume any warranty for the appliance Disposal.

Disposal

- Please dispose of the disinfectant at your local recycling stations in the containers provided.

- Unused residues of cleaning agents and disinfectants must be disposed of in accordance with the information in the safety data sheet and must not simply be removed via the drain and sewage system.

Scope of delivery

- 1x Difenser main pillar
- 1x Stand
- 1x Power supply plug 90-230V-50Hz/60Hz *
- 2x Canister lid
- 1x 5 Liter return container
- 1x Key for main pillar
- 1x Display board (optional)
- 1x Anti-theft protection

*The power cord is available with the following plug types: EU, GB, USA

Technical Data

Power supply voltage:	90V – 230V / 50Hz -60Hz
Battery voltage:	24V / 15Ah / charging current 1,5A
Power consumption:	approx. 90Watt
Output volume:	0,5ml – 3ml
Disinfectant:	up to 10 Liter
Return container:	5 Litre
Dimensions in mm (L x W x H):	250 x 250 x 1200
Dimensions stand in mm:	450 x 450
Weight (without Disinfectant)	
Basic [PSU]:	26 kg
Basic Battery [with PSU & Battery]:	35 kg

Conformity:

RoHS compliant
Machinery directive 2009 / 127 / EC
EMC- directive 2004 / 30 / EC

Mounting of the stand | Anti-theft protection

- Mounting the stand (see fig.1) under the main pillar. Please use the four M6x16 hexagon socket screws. For mounting you need a hexagon wrench size 4.

fig. 1

- To use the supplied anti-theft protection, simply insert the lock into the opening provided in the base plate and lock it by turning it using the supplied key (fig. 2 & 3).

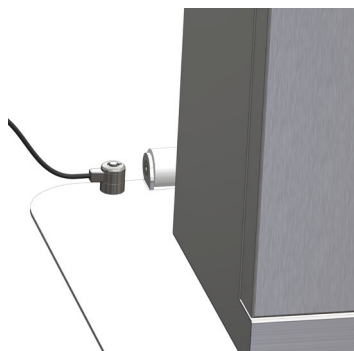
fig. 2




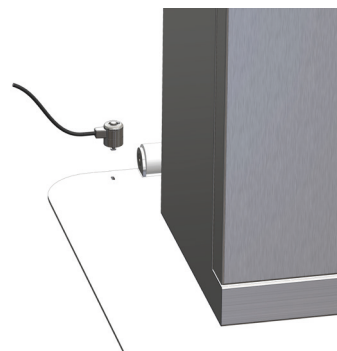
fig. 3

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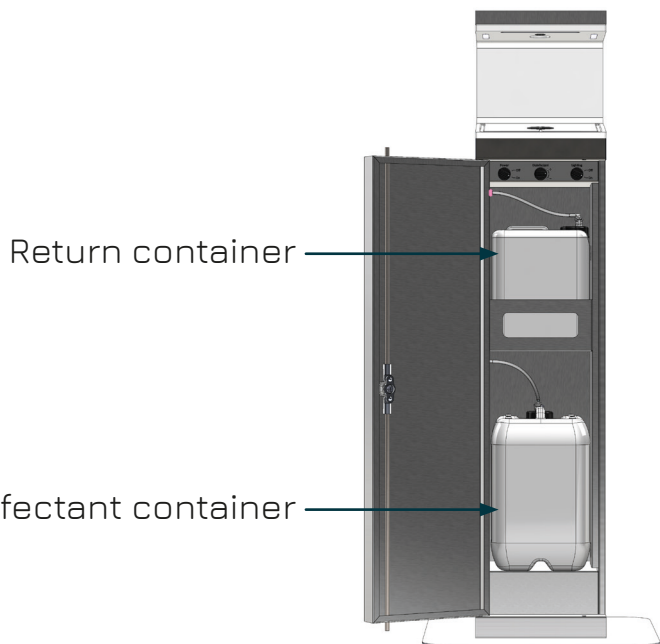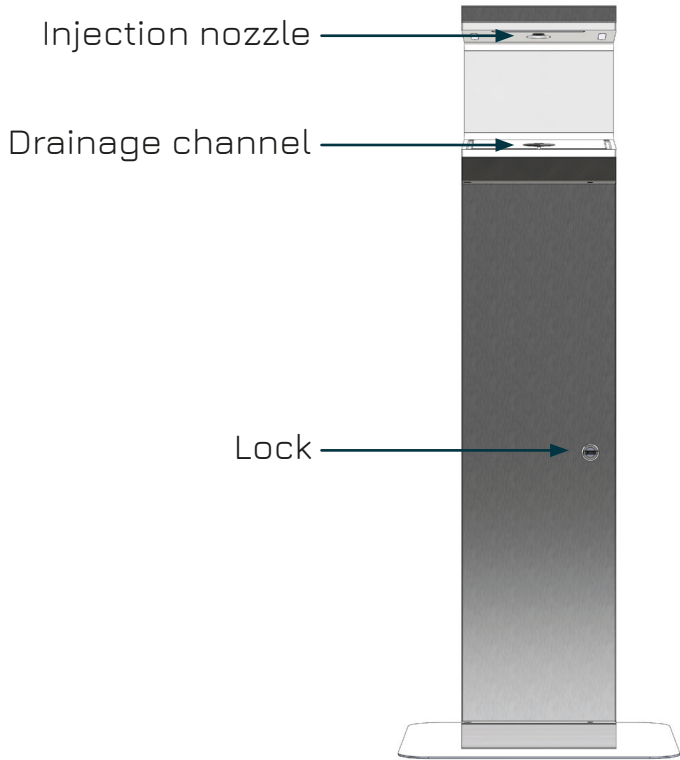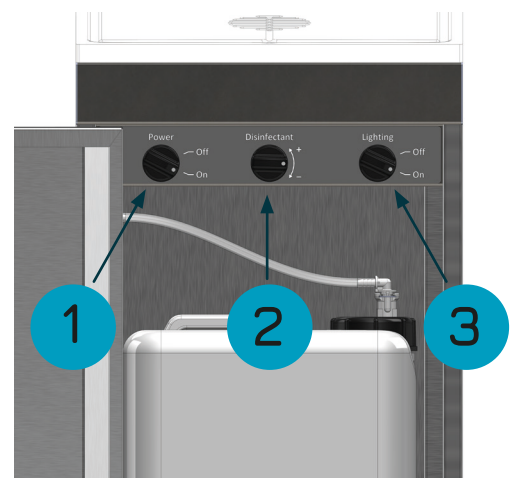Commissioning

- If you have the battery version of the appliance, please note that the batteries are not fully charged and must first be charged for operation.
- Now select the appropriate location for the appliance, taking into account the safety instructions.
- Use the key provided to open the front door of the appliance and place the container with disinfectant in the space provided [see Overview].
- Plug the power cord into the power socket at the back of the appliance.
- Now set the rotary switch labeled **Power** to ON [see overview]. The appliance is now ready for operation.
- Use the **Disinfectant** rotary switch to set the desired disinfectant output volume in 6 steps from 0.5ml - 3ml [see overview]. The RKI recommends a target quantity of 3ml.
- With the **Lighting** rotary switch you can optionally switch on the LED basic lighting.

Operation

- Non-contact insertion of hands into the opening, both palms open upwards. Fingertips slightly angled upwards.
- Via sensor recognition, the palms are automatically sprayed with disinfectant. The spray quantity is adjustable variably [see.commissioning].
- Remove your hands from the appliance and rub the hand disinfectant for approx. 30 seconds for optimum elimination of pathogens [if necessary, see the application panel on the appliance].

Operating conditions

- Flashing [red] regularly
-> Disinfection container empty, please fill it up again.
- Fast [red] flashing
-> return container full, please empty/disposal properly.
- Permanent [red] light
-> safety shutdown, object / dirt in front of the sensor.
- Alternating [red/blue] light (only available in the battery version)
-> Battery voltage low [approx 25%], please charge the battery.

Maintenance | attendance

- Longer periods of non-use do not affect the appliance, but please note that the batteries must be recharged regularly.
- The Difenser is a premium product, so clean the outside of the appliance only with a stainless steel cleaner. If you have a powder-coated colour version, use a commercially available household cleaner / equipment cleaner, not acidic or chlorinated cleaners.
- Do not spray with water, do not use compressed air or solvents to remove dirt or residues.
- Do not disassemble the appliance.

The Battery pack

- The built-in battery pack consists of two maintenance-free 12V lead gel batteries with 14Ah.
- The battery pack is charged via the mains plug when the Power rotary switch is set to OFF. The charging time for discharged batteries is approx. 10 hours. Please never store the appliance with a discharged battery for a longer period of time, this can lead to a loss of capacity.
- If handled correctly, the battery pack retains its capacity for several years or several hundred charge/discharge cycles.

Technical Data:

Model:	lead-gel accumulator
Output voltage:	12V/ 14 Ah per accumulator
Charging current:	1,25A
Temperature range:	5°C to 45°C
Dimensions in mm [L x W x H]:	151 x 98 x 97,5
Weight:	4,2 kg

Dilizer GbR
Bünteweg 33
30989 Gehrden
Germany

info@dilizer.com
www.dilizer.com